

MY SURECLICK® JOURNEY RESOURCE

A RESOURCE ROADMAP TO HELP SUPPORT YOUR SELF-INJECTION JOURNEY

This resource is intended to provide you with an overview and basic information about the SureClick® Autoinjector (140 mg/mL prefilled single-dose autoinjector) and does not replace the Instructions for Use. Please review the Instructions for Use included with your prescribed medication prior to performing your injection. If you have further questions, please contact your doctor.

APPROVED USE

What is Repatha®?


Repatha® is an injectable prescription medicine used:

- to reduce the risk of major adverse cardiovascular (CV) events, such as death from cardiovascular disease, heart attack, stroke, certain types of chest pain conditions (unstable angina) requiring hospitalization, or certain types of heart surgery, in adults at increased risk for these events.
- Along with diet and exercise to reduce low-density lipoprotein (LDL) or bad cholesterol in adults with high blood cholesterol levels called hypercholesterolemia and in adults with a type of high cholesterol called heterozygous familial hypercholesterolemia (HeFH).

IMPORTANT SAFETY INFORMATION

Do not use Repatha® if you are allergic to evolocumab or to any of the ingredients in Repatha®.

[Please see Important Safety Information on pages 23-24. Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

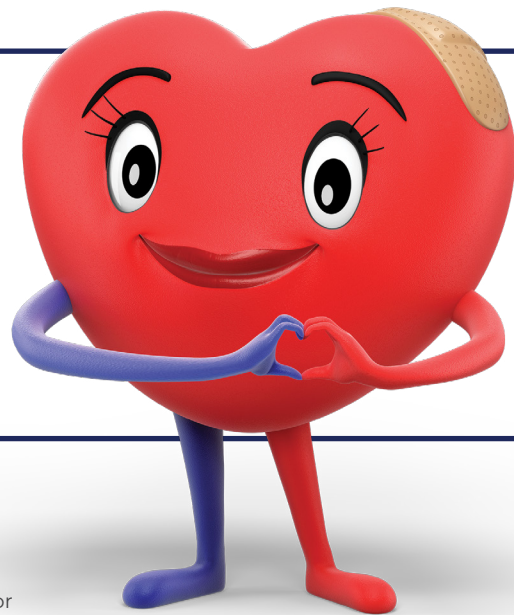

 **Repatha®**
(evolocumab) injection
140 mg/mL

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE


IMPORTANT
SAFETY INFORMATION

GETTING STARTED WITH YOUR SURECLICK® AUTOINJECTOR

This resource is intended to provide you with an overview and basic information about the SureClick® Autoinjector and does not replace the Instructions for Use. Please review the Instructions for Use included with your prescribed medication prior to performing your injection. If you have further questions, please contact your doctor.

This resource roadmap may help you with questions about using the SureClick® Autoinjector.

Scan the code to view the Instructions for Use

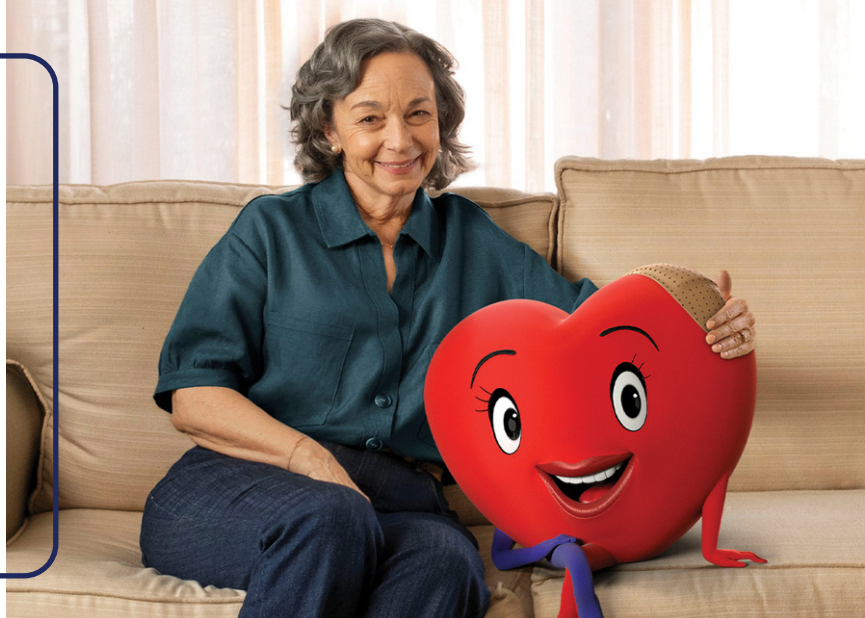

IMPORTANT SAFETY INFORMATION (CONT'D)

Before you start using Repatha®, tell your healthcare provider about all your medical conditions, including if you are allergic to rubber or latex, are pregnant or plan to become pregnant, or are breastfeeding or plan to breastfeed. Repatha® is available as prefilled single-dose SureClick® autoinjectors and prefilled single-dose syringes that either contain dry natural rubber (a derivative of latex) in the needle cover or are not made with natural rubber latex. The carton and "Instructions for Use" in the carton will state if your prefilled single-dose SureClick® autoinjector or prefilled single-dose syringe contains dry natural rubber. The single-dose Pushtronex® system (on-body infusor with prefilled cartridge) is not made with natural rubber latex.

Tell your healthcare provider or pharmacist about any prescription and over-the-counter medicines, vitamins, or herbal supplements you take.

[Please see Important Safety Information on pages 23-24. Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

GETTING STARTED AT HOME WITH YOUR SURECLICK® AUTOINJECTOR

When you start using the SureClick® Autoinjector for the very first time, you may wonder if you're going to do it right.

You may have questions or feel uncertain about the process. Other patients feel the same. You are not alone.

There's a lot to learn, and you want to get it right. We're here to support you along the way.

Self-injection can become a simple routine.



With this resource and the online tools, you can do this...

Now, let's get started!

Carefully review this educational booklet which offers additional helpful information just for you and your caregiver, including:

1. Key injection steps
2. Helpful tips
3. Resources and tools
4. Answers to frequently asked questions



FIND A QUIET, CALM PLACE TO INJECT

Start off right and don't rush

You may want to find a quiet and calm place in your home, where pets or other people can't distract or interrupt you. Consider a room where you can close the door. Sit in a comfortable chair so you feel more relaxed.

This will soon become part of your routine. Gather your materials and place on a clean, flat, and well-lit surface.

- Device at room temperature
- Sharps disposal container
- Alcohol wipe
- Adhesive bandage
- Cotton balls or gauze pads



Please see [Important Safety Information](#) on pages 23-24. [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

INJECT

In this section, we will discuss the steps for injecting the medication.



Please read the Instructions for Use for detailed information prior to starting the injection.

5

NEED HELP?  Call 1-844-REPATHA (1-844-737-2842), Monday - Friday, 8:00 am to 8:00 pm ET.



Visit repatha.com/injection

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

GET TO KNOW YOUR SURECLICK® AUTOINJECTOR

Introduction to the Autoinjector

- SureClick® Autoinjector is a ready-to-use Autoinjector prefilled with your medication.
- Please read the Instructions for Use for detailed information prior to performing your injection.

Inspect the Medication

- Make sure the medication in the window is clear and colorless to slightly **yellow**.
- Check Expiration Date: **DO NOT** use the Autoinjector after the expiration date on the label.

⚠ DO NOT use the Autoinjector if the carton is damaged or the seal is broken.

⚠ DO NOT remove cap until ready to inject. Inject within **5 minutes** of removing cap.

Please see [Important Safety Information on pages 23-24](#). [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)


Key Parts of the Autoinjector

GRAY START BUTTON will work only after the **YELLOW SAFETY GUARD** has been unlocked.


LOT/EXPIRATION DATE

WINDOW will turn **yellow** during the injection

YELLOW SAFETY GUARD unlocks the start button and covers the needle after injection is complete

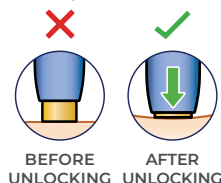

⚠ DO NOT touch safety guard (needle inside).

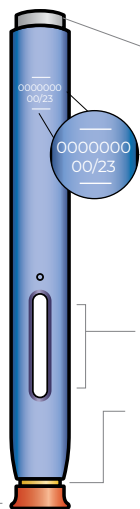

PREP | READY THE AUTOINJECTOR

Prep the Autoinjector: Inspect it and bring it to room temperature naturally.



Take out one Autoinjector that you need for the current injection.

- Gather supplies, including sharps container.
 - **IMPORTANT:** Check the Autoinjector label and prescription to make sure you have the correct medicine and dose.
- ⚠ Keep the Autoinjector and all medicines out of the reach of children.



Allow the Autoinjector to come to room temperature naturally for at least 30 minutes.

- Autoinjector can be kept at room temperature between 68°F to 77°F (20°C to 25°C) for up to 30 days.



Inspect the medication

- **DO NOT** use the Autoinjector if the medicine is cloudy, discolored, or contains flakes or particles.
- **DO NOT** use the Autoinjector if the orange cap is missing or loose in carton.

⚠ **DO NOT** put back in refrigerator once at room temperature.

⚠ Discard medication if left out of the refrigerator after 30 days.

⚠ **DO NOT** leave in car.

⚠ **DO NOT** leave the Autoinjector in direct sunlight.

⚠ **DO NOT** warm or heat the Autoinjector.


⚠ **DO NOT** freeze or use the Autoinjector if it has been frozen.



PREP | SELECT AND CLEAN THE INJECTION SITE

Select your injection site

- Select one of these injection sites.
 - Stomach (except for a 2-inch area around your belly button)
 - Thigh
- Someone else can inject in your thigh, stomach, or back of your upper arm

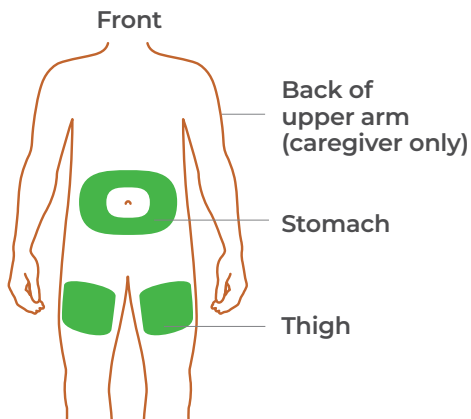

Important!

Be sure to change your injection site each time to avoid skin irritation.

⚠ DO NOT inject into areas with scars or stretch marks, or where the skin is tender, bruised, red, hard, raised, thick or scaly skin patch, or lesion.

Prep your injection site

- Wash your hands with soap and water.
- Clean the injection site with an alcohol wipe, and let your skin dry on its own.

⚠ DO NOT touch the cleaned injection site again before injecting.

Tips!

- Consider an injection site that has a thicker fatty layer.
- Some people keep a diary or log to track their injections.

Please see [Important Safety Information](#) on pages 23-24. [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

INJECT | REMOVE THE ORANGE CAP


Pull the orange cap straight off only when you can inject right away. You may need to pull hard.

- If you have questions, seek help by referencing the Instructions for Use or calling 1-844-REPATHA (1-844-737-2842) before removing the cap.



Use the Autoinjector within 5 minutes after removing the cap.

- Waiting longer than 5 minutes can dry out the medicine.

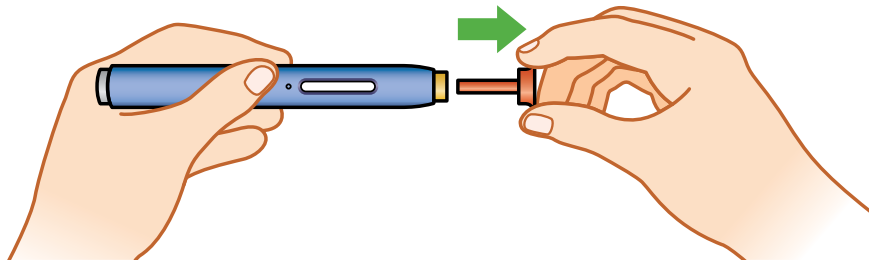

- ⚠ **DO NOT** twist, bend, or wiggle the orange cap to pull it off.
- ⚠ **DO NOT** take the orange cap off until you can inject right away, in order to prevent the medication from drying out.
- ⚠ **DO NOT** put the cap back onto the Autoinjector once it has been removed.

Tip!

It is normal to see a drop of medicine come out of the needle or **yellow** safety guard.

9

NEED HELP? 📞 Call 1-844-REPATHA (1-844-737-2842), Monday - Friday, 8:00 am to 8:00 pm ET.



Visit repatha.com/injection

GETTING
STARTED

INJECT

RESOURCES


FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION


INJECT | PINCH OR STRETCH THE SKIN TO CREATE A FIRM SURFACE AT THE INJECTION SITE UNTIL THE INJECTION IS FINISHED

STRETCH

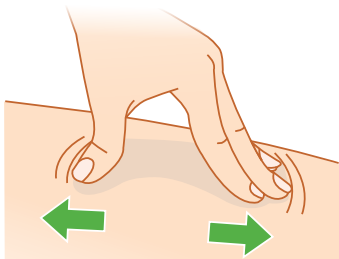

Stretch the skin firmly by moving your thumb and fingers in opposite directions, creating a firm surface.

PINCH

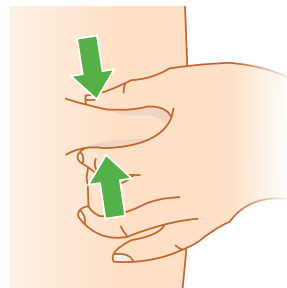

Pinch the skin firmly between your thumb and fingers, creating a firm surface.

 It is important to keep the skin stretched or pinched throughout the injection.

Please see [Important Safety Information](#) on pages 23-24. [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

10

NEED HELP?  Call 1-844-REPATHA (1-844-737-2842), Monday - Friday, 8:00 am to 8:00 pm ET.



Visit repatha.com/injection

GETTING
STARTED

INJECT


RESOURCES

FAQ


TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

INJECT | PLACE THE SAFETY GUARD STRAIGHT DOWN ON THE SKIN

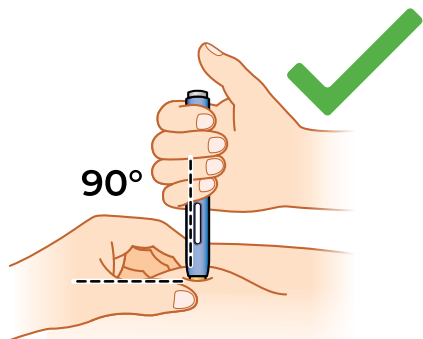


Place Autoinjector **straight down** onto the skin at a 90-degree angle.

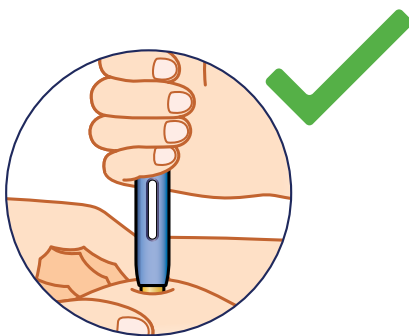

Be sure you can see the entire medication window and that your fingers don't cover the window.

⚠ DO NOT change positions after the injection has begun.

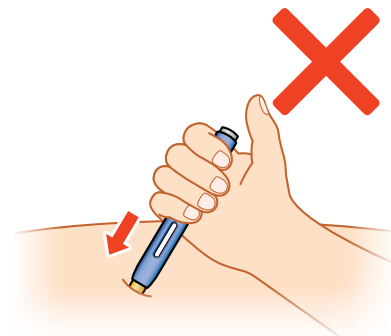


⚠ DO NOT apply the Autoinjector to the skin at an angle. You may not be able to make full contact with the skin, and that can prevent the Autoinjector from working properly.

Please see Important Safety Information on pages 23-24. [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

INJECT | PUSH AUTOINJECTOR FIRMLY DOWN ONTO THE SKIN

To unlock the gray start button, **PUSH** the Autoinjector firmly onto the skin until you can't push it anymore.

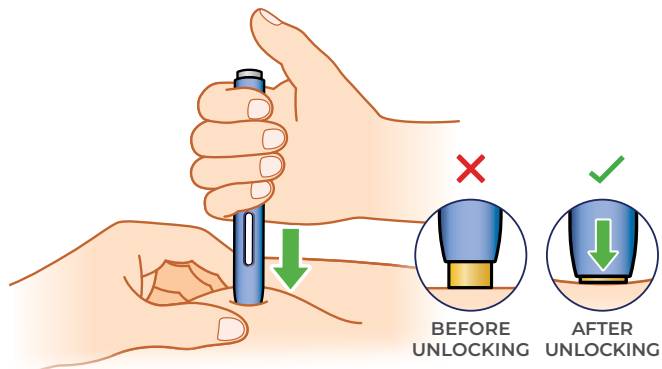


PUSH the Autoinjector firmly down until the **yellow** safety guard pushes onto the skin and stops moving.

- This unlocks the gray start button.
- **KEEP PUSHING. DO NOT LIFT.**
- Gray start button will only work after the **SAFETY GUARD** has been pushed all the way in.

INJECT | PRESS GRAY START BUTTON TO BEGIN INJECTION

Next, **PRESS** the gray start button while pushing the Autoinjector down firmly.

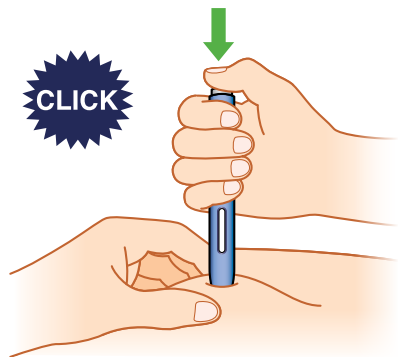

PRESS start button until you hear or feel a click.


- **KEEP PUSHING** the Autoinjector down with the same pressure throughout the injection.
- The window will start to turn **yellow**.
- It is okay to let go of the gray start button.

- ⚠ REMEMBER:** You **MUST** unlock the Autoinjector by firmly pushing onto your skin **WHILE** pressing the start button to perform the injection.
- ⚠** If you do not hear a click after pressing the start button, and the window is still clear, ensure that the safety guard is pushed all the way down first. Then try to press the button again.
- ⚠** These steps need to be completed in this order for a successful injection.

Please see [Important Safety Information on pages 23-24](#). [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

INJECT | WATCH MEDICATION WINDOW TURN FULLY YELLOW

Finally, watch the medication window turn **YELLOW** to indicate the successful completion of the injection.

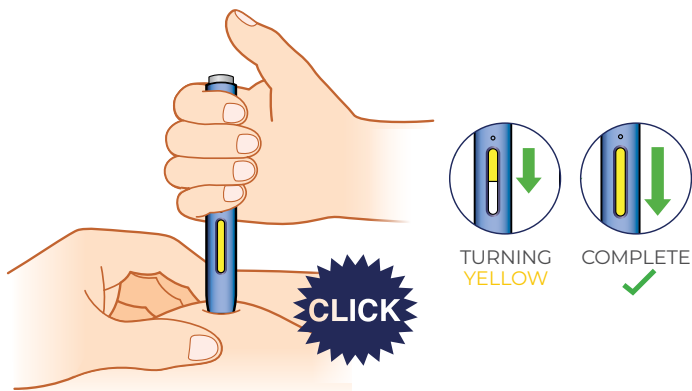

WATCH window turn fully **yellow** to ensure all the medication has been injected.

- The injection may take up to **15 seconds** to complete.
- You may hear or feel a click.
- After the window turns fully **yellow**, lift the autoinjector away from the skin.
- Confirm a full dose of medicine was injected.
- Do not touch the **yellow** safety guard.
- A small drop of liquid on the injection site is okay.

★ **CONGRATULATIONS** on reviewing the injection process!

Continue to practice the steps **PUSH, PRESS, & WATCH** to help build confidence. Start to make these simple steps a part of your routine, and be sure to contact your care team if you have any questions or concerns.

14

NEED HELP?  Call 1-844-REPATHA (1-844-737-2842), Monday - Friday, 8:00 am to 8:00 pm ET.



Visit repatha.com/injection

GETTING
STARTED

INJECT

RESOURCES



FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

STORAGE AND DISPOSAL

Refrigeration guidelines:

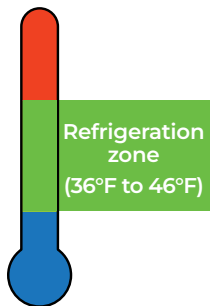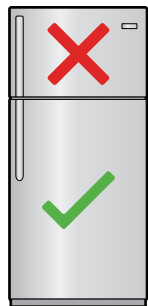

⚠ **DO NOT** freeze Autoinjector.

- When you receive the Autoinjector, be sure to place it directly into the refrigerator.
- Store Autoinjector in the refrigerator (36°F to 46°F) until ready to use.
- Store Autoinjector in original carton to protect from light or physical damage.
- Only remove the Autoinjector you plan to use.

Tip!

Call **1-844-REPATHA** to replace your sharps container.


Disposal guidelines:

- Put the used Autoinjector and cap in an FDA-cleared sharps container right away after use.

⚠ **DO NOT** throw away the Autoinjector or cap in your household trash.

⚠ **DO NOT** put the orange cap back on.

⚠ **DO NOT** touch the yellow safety guard.

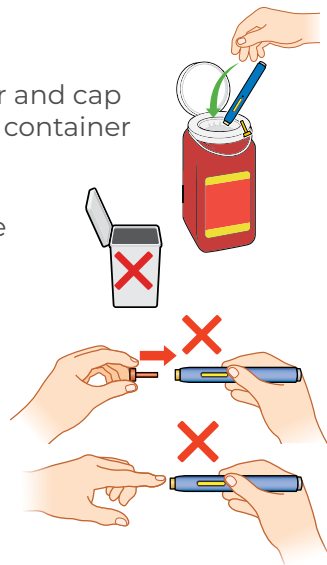

Please see Important Safety Information on pages 23-24. [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

CONGRATULATIONS

on reviewing the injection process!

Continue to practice the steps at home. **PUSH, PRESS, & WATCH**... to help build confidence. Start to make these simple steps a part of your routine, and be sure to contact your care team if you have any questions or concerns.



IMPORTANT SAFETY INFORMATION (CONT'D)

What are the possible side effects of Repatha®?

Repatha® can cause serious side effects including serious allergic reactions. Stop taking Repatha® and call your healthcare provider or seek emergency help right away if you have any of these symptoms: trouble breathing or swallowing, raised bumps (hives), rash or itching, swelling of the face, lips, tongue, throat or arms.

Please see Important Safety Information on pages 23-24. [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

16

NEED HELP?  Call 1-844-REPATHA (1-844-737-2842), Monday - Friday, 8:00 am to 8:00 pm ET.



Visit repatha.com/injection

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

RESOURCES

In this section, we will discuss a variety of helpful resources available to you that are just a click or phone call away.



Please see Important Safety Information on pages 23-24. [Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

17

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

SURECLICK® AUTOINJECTOR PATIENT SUPPORT IS AVAILABLE

If you have any questions along the way, support is just a phone call or a few clicks away. Call 1-844-REPATHA (1-844-737-2842), Monday - Friday, 8:00 am to 8:00 pm ET, or visit repatha.com/injection anytime for:

Live Support



Call 1-844-REPATHA
(1-844-737-2842),
Monday - Friday,
8:00 am to 8:00 pm ET.

Injection Overview Video



Hold smartphone
camera over QR code

Additional Resources



repatha.com/injection



FAQ

TEMPERATURE CONTROL AND STORAGE:

What is the best way to store my SureClick® Autoinjector? Store in the refrigerator (36°F to 46°F, 2°C to 8°C) in its original carton.

⚠ DO NOT freeze.

I left my Autoinjector outside of the refrigerator. Is this okay? Your Autoinjector can be stored at room temperature (68°F to 77°F, 20°C to 25°C) for up to **30 days**. Do not return to refrigerator once it reaches room temperature.

Room temperature:
68°F to 77°F,
20°C to 25°C




ROOM TEMPERATURE GUIDELINES:

Do I really have to wait 30 minutes for the Autoinjector to reach room temperature before injecting? Yes.

This is important for administering the entire dose and helps minimize discomfort. Repatha® may take longer to inject if it has not reached room temperature. Do not heat the Autoinjector. Let it come to room temperature naturally.

Why do I have to use the Autoinjector within 5 minutes after I remove the cap? If you wait longer than 5 minutes, the medication can dry out and clog the needle, resulting in failure to start injection.

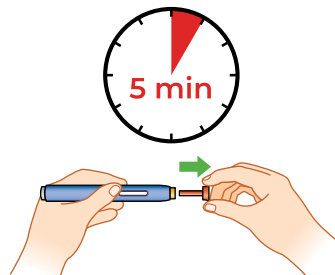

[Please see Important Safety Information on pages 23-24. Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

FAQ

INJECTION SITE QUESTIONS:

Why do I need to stretch or pinch the skin? You need to **STRETCH** or **PINCH** the skin to create an area 2 inches wide **to maintain a firm surface** throughout the entire injection.

Can I move the Autoinjector around on my skin? Yes, it's okay to move **BEFORE** you **PUSH** the start button.


⚠ DO NOT move the Autoinjector **AFTER** you **PUSH** the **START** button, as you may not receive the full dose of medication.

START BUTTON QUESTIONS:

Do I *press and release* the start button, or do I *press and hold down* the start button during the injection? If you need to lift your thumb, it's okay. Avoid lifting the Autoinjector until the injection is complete.

What will happen if I press the start button before I am ready to do the injection? The Autoinjector will not work and will not dispense medication unless the safety guard is unlocked prior to the start button activation.

What if I press the start button, but it doesn't click? Make sure you push the Autoinjector down so that the safety guard pushes in and stops moving. If the safety guard is not depressed fully, the start button may not work. You will hear a **CLICK** when the start button has been activated correctly.

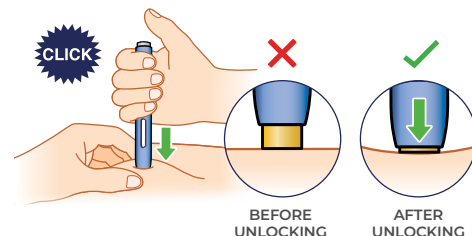

GENERAL TRAVEL GUIDANCE

GENERAL TRAVEL GUIDANCE

Yes, you can travel. You don't have to miss any of your treatments. In fact, it's important that you don't miss your injection when traveling.

- Always keep your medication with you to ensure it is kept within the safe temperature zone and is not broken.

⚠ DO NOT store the Autoinjector in extreme heat or cold. For example, avoid storing the Autoinjector in your vehicle's glove box or trunk.

⚠ DO NOT shake the Autoinjector.

⚠ DO NOT use the Repatha® SureClick® Autoinjector if it has been dropped on a hard surface. Part of the Repatha® SureClick® Autoinjector may be broken even if you cannot see the break. Use a new Repatha® SureClick® Autoinjector, and call 1-844-REPATHA (1-844-737-2842).

[Please see Important Safety Information on pages 23-24. Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

SHORT TRIPS

- Autoinjector can be kept at room temperature at 68°F to 77°F (20°C to 25°C) up to 30 days without refrigeration.

⚠ Remember that once your Autoinjector reaches room temperature, you should not put it back in the refrigerator.

Important: Keep Autoinjector in original carton.



LONGER TRIPS


- Reach out to your pharmacist and/or doctor to discuss other options such as filling your prescription at your desired destination.



CAR AND AIR TRAVEL GUIDANCE

It's okay to travel via car:


- ⚠️ **DO NOT** leave Autoinjector in car or exposed to extreme heat, extreme cold, or direct sunlight.
- ⚠️ **DO NOT** store the Autoinjector in your vehicle's glove box or trunk.
- Once Autoinjector reaches room temperature, do not put back in the refrigerator.

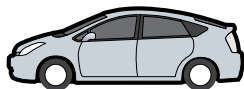

Traveling by airplane:

Before you travel

- Contact your airline about the carry-on policy for syringes or injection devices. Security regulations may change.
- Research the injection device requirements of other countries you may visit.
- Bring a copy of your prescription or a doctor's note to show airport security.

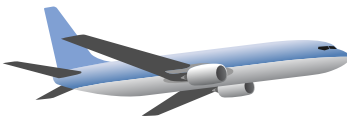

TSA or airport security considerations:

- Notify security screeners that you are carrying a sharps container.
- Consider requesting a “hand check” of your medication kit to avoid any delays.
- Be prepared to display, handle, and repack your own medication and supplies in order to prevent contamination or damage.



IMPORTANT SAFETY INFORMATION

Do not use Repatha® if you are allergic to evolocumab or to any of the ingredients in Repatha®.

Before you start using Repatha®, tell your healthcare provider about all your medical conditions, including if you are allergic to rubber or latex, are pregnant or plan to become pregnant, or are breastfeeding or plan to breastfeed. Repatha® is available as prefilled single-dose SureClick® autoinjectors and prefilled single-dose syringes that either contain dry natural rubber (a derivative of latex) in the needle cover or are not made with natural rubber latex. The carton and “Instructions for Use” in the carton will state if your prefilled single-dose SureClick® autoinjector or prefilled single-dose syringe contains dry natural rubber.

The single-dose Pushtronex® system (on-body infusor with prefilled cartridge) is not made with natural rubber latex.

Tell your healthcare provider or pharmacist about any prescription and over-the-counter medicines, vitamins, or herbal supplements you take.

What are the possible side effects of Repatha®?

Repatha® can cause serious side effects including serious allergic reactions. Stop taking Repatha® and call your healthcare provider or seek emergency help right away if you have any of these symptoms: trouble breathing or swallowing, raised bumps (hives), rash or itching, swelling of the face, lips, tongue, throat or arms.

The most common side effects of Repatha® include: runny nose, sore throat, symptoms of the common cold, flu or flu-like symptoms, back pain, high blood sugar levels (diabetes) and redness, pain, or bruising at the injection site.

Tell your healthcare provider if you have any side effect that bothers you or that does not go away.

These are not all the possible side effects of Repatha®. Ask your healthcare provider or pharmacist for more information. Call your healthcare provider for medical advice about side effects.

[Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.](#)

IMPORTANT SAFETY INFORMATION (CONT'D)

You are encouraged to report negative side effects of prescription drugs to the FDA. Visit www.fda.gov/medwatch, or call 1-800-FDA-1088.

APPROVED USE

What is Repatha®?

Repatha® is an injectable prescription medicine used:

- to reduce the risk of major adverse cardiovascular (CV) events, such as death from cardiovascular disease, heart attack, stroke, certain types of chest pain conditions (unstable angina) requiring hospitalization, or certain types of heart surgery, in adults at increased risk for these events.

- Along with diet and exercise to reduce low-density lipoprotein (LDL) or bad cholesterol in adults with high blood cholesterol levels called hypercholesterolemia and in adults with a type of high cholesterol called heterozygous familial hypercholesterolemia (HeFH).

Please see full Prescribing Information, including Patient Product Information.

Please see Important Safety Information on pages 23-24. Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.

NEED HELP?  Call 1-844-REPATHA (1-844-737-2842), Monday - Friday, 8:00 am to 8:00 pm ET.



Visit repatha.com/injection

GETTING
STARTED


INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

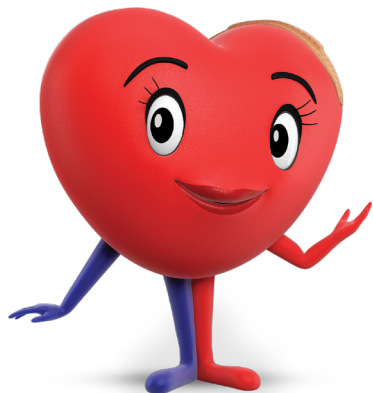

**IF YOU NEED SUPPORT,
CONTACT YOUR HEALTHCARE PROVIDER, OR:**

- **CALL 1-844-REPATHA (1-844-737-2842),
Monday - Friday, 8:00 am to 8:00 pm ET.**
- **VISIT repatha.com/injection**

APPROVED USE

What is Repatha®?

Repatha® is an injectable prescription medicine used:

- to reduce the risk of major adverse cardiovascular (CV) events, such as death from cardiovascular disease, heart attack, stroke, certain types of chest pain conditions (unstable angina) requiring hospitalization, or certain types of heart surgery, in adults at increased risk for these events.

IMPORTANT SAFETY INFORMATION

Do not use Repatha® if you are allergic to evolocumab or to any of the ingredients in Repatha®.

Please see Important Safety Information on pages 23-24. Click here for the full Prescribing Information, Patient Product Information and Instructions for Use.



© 2022-2024, 2026 Amgen Inc.
All rights reserved. Not for Reproduction. USA-CCF-84108 03/26



**GETTING
STARTED**

INJECT

RESOURCES

FAQ

**TRAVEL
GUIDANCE**

**IMPORTANT
SAFETY INFORMATION**